# AIR-FLOW® PERIO

## PROPHYLAXIS POWDER

Before using this product, please carefully read and follow the recommendations of the instruction manuals originally delivered with your device and system. Please pay special attention to the safety precautions.

### Product Description

The AIR-FLOW® Perio (prophylaxis powder) is used for subgingival treatments.

This powder is for use in dentistry with the Hu-Friedy & EMS air polishing devices.

### Indications for use

The AIR-FLOW® Perio Prophylaxis Powder is indicated for use with EMS dental handpieces for cleaning and preparation of teeth.

### Contra-indications

- ⚠ Patients suffering from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.
- ⚠ Predisposed persons may be sensitive to glycooll. If allergic reactions are observed, stop using the product and completely remove the powder from mouth.
- ⚠ Note that powder is provided non-sterile. Please take this information into consideration when performing procedures. The powder should only be used for non-surgical procedures.
- ⚠ The treatment of deep periodontal pockets can cause bacteraemia. Please apply appropriate restrictions for the treatment of risk patients (general weakened immunity, endocarditis).
- ⚠ Do not direct the jet of powder toward fillings, crowns or bridgework as this could damage these restorations.

### Recommendations for the treatment

- ⚠ Patient and user have to wear eye protection.
- ⚠ User has to wear a mask and gloves.
- ⚠ Use the high-speed evacuator of your dental unit.
- ✋ Calculi and tartar cannot be removed with this powder.
- ✋ Apply Vaseline® on the patient's lips.
- ✋ Wash hands after use.
- ✋ Wash patient face after use (e.g. with a wet wipes).

### General information

- Do not exceed the powder filling limit as indicated on the container.
- This powder is sugar free.

### Storage and shelf-life

- The powder should not be stored near heat sources.
- Storage condition up to 25°C and 10% to 90% relative humidity.
- Transport conditions -29°C to 38°C and 10% to 90% relative humidity
- Do not use after expiration date.

### Also available:

- AIR-FLOW® CLASSIC available in Lemon, Mint, Cassis, Cherry, Tropical and Neutral (for supragingival treatments).
- AIR-FLOW® CLASSIC COMFORT (for supragingival treatments).
- AIR-FLOW® SOFT (salt free for supragingival treatments).
- AIR-FLOW® PLUS (salt free for subgingival and supragingival treatments).

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